


Edwards Lifesciences™

**Edwards FloTrac Sensor  
&  
Edwards Vigileo Monitor**

*Understanding Stroke Volume Variation  
and Its Clinical Application*

Edwards FloTrac System



Edwards


The image shows the front panel of the Edwards Vigileo monitor on the left, featuring a color LCD screen displaying a large '82' and a waveform. To the right is the FloTrac sensor, a small white device with a blue cap and a white cable. The background is a light, neutral color with a subtle green wave graphic.



## Topics

- **System Configuration**
- **Pulsus Paradoxes**
- **Reversed Pulsus Paradoxus**
- **What is Stroke Volume Variation (SVV)?**
- **How is SVV Calculated?**
- **Sensitivity and Specificity**
- **Clinical Application of SVV**
- **Limitations of SVV**

## System Configuration

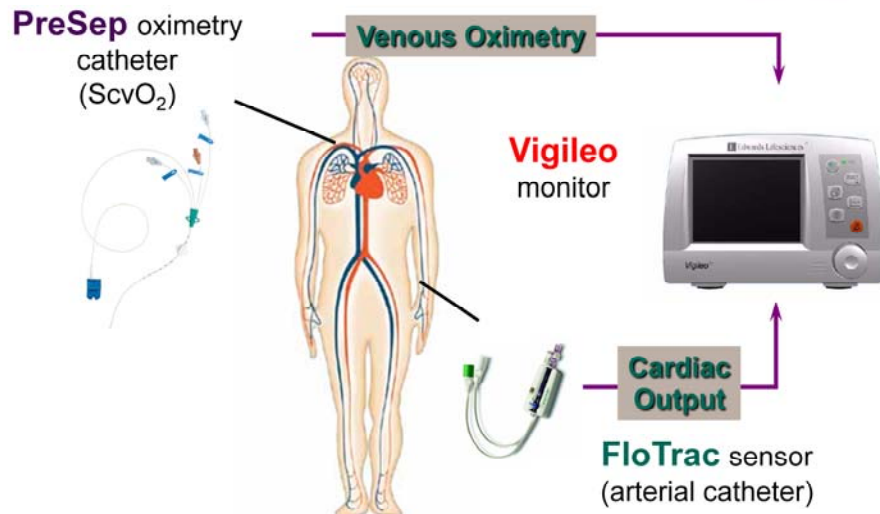

 Edwards Lifesciences

The Vigileo monitor by Edwards Lifesciences supports both the FloTrac Sensor for continuous cardiac output and the Edwards PreSep oximetry catheter for continuous central venous oximetry (ScvO<sub>2</sub>)

## Vigileo monitor

- Continuously computes stroke volume from the patient's arterial pressure signal
- Displays key hemodynamic parameters on a continuous basis (every 20 sec)
- Requires **NO** manual calibration
  - The user simply enters patient age, gender, height and weight to initiate monitoring
  - Advanced waveform analysis compensates for:
    - Patient-to-patient differences in vasculature
    - Real time changes in vascular tone
    - Differing arterial sites
- Venous oximetry available when used with appropriate Edwards oximetry catheters



 Edwards Lifesciences

The Vigileo monitor continuously displays and updates Continuous Cardiac Output, Cardiac Index, Stroke Volume, Stroke Volume Index, Systemic Vascular Resistance\*, Systemic Vascular Resistance Index\*, and Stroke Volume Variation every 20 seconds when used with the FloTrac Sensor. DO<sub>2</sub> and DO<sub>2</sub>I are also available for manual calculation.\*\* These parameters help guide the clinician in optimizing stroke volume through precision guided management of preload, afterload, and contractility.

Vascular tone = vessel compliance and resistance

The Vigileo monitor then helps identify the adequacy of cardiac output by monitoring central venous (ScvO<sub>2</sub>) or mixed venous (SvO<sub>2</sub>) oxygen saturation when used with Edwards venous oximetry technologies.

*\* These parameters require the CVP value to be slaved from bedside monitor for continuous monitoring. SVR/SVRI can also be assessed on the Derived Value Calculator for intermittent calculations using either slaved or manually entered MAP, CVP, and CO values.*

*\*\* These parameters require the SpO<sub>2</sub> and PaO<sub>2</sub> values to be manually entered. If CO is being continuously monitored, the calculator will default to the existing CO value. Otherwise, the user may override the continuous value to manually enter CO.*

## FloTrac sensor



The specially designed **FloTrac** sensor provides the high fidelity arterial pressure signal required by the Vigileo monitor to calculate the stroke volume

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
 Edwards Lifesciences

The Vigileo monitor uses the patient's arterial pressure waveform to continuously measure cardiac output. With inputs of height, weight, age and gender, patient-specific vascular compliance is determined.

The FloTrac Sensor measures the variations of the arterial pressure which is proportional to stroke volume. Vascular compliance and changes in vascular resistance are internally compensated for.

Cardiac output is displayed on a continuous basis by multiplying the pulse rate and calculated stroke volume as determined from the pressure waveform.

The FloTrac Sensor is easily setup and calibrated at the bedside using the familiar skills used in pressure monitoring.





## Normal Variation & Pulsus Paradoxus

In a normal individual who is breathing spontaneously, blood pressure decreases on inspiration

The exaggeration of this phenomenon is called pulsus paradoxus

*F. Michard*

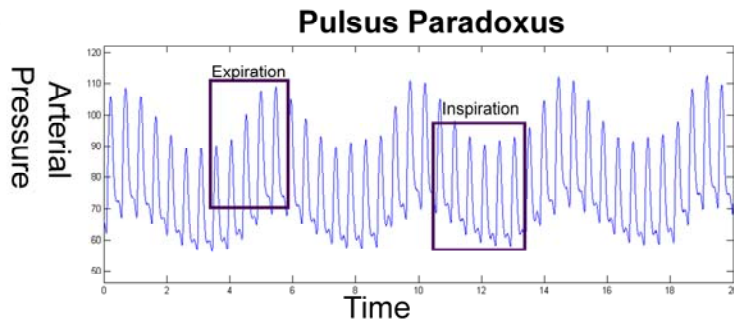

 Edwards Lifesciences


In a normal individual who is breathing spontaneously, blood pressure decreases on inspiration. The range of normal peak decreases in systolic pressure have been reported between 5 – 10 mmHg.

The exaggeration of this phenomenon, called pulsus paradoxus, was initially reported by Adolf Kussmaul in constrictive pericarditis and was described as a “pulse disappearing during inspiration and returning during expiration” despite the continued presence of the cardiac activity during both respiratory phases.

**Michard F. Changes in Arterial Pressure During Mechanical Ventilation. *Anesthesiology* 2005; 103:419-28.**

**Swami A. Pulsus Paradoxus in Cardiac Tamponade: A Pathophysiologic Continuum, *Clinical Cardiology* 2003; 26, 215-217**

## Reversed Pulsus Paradoxus

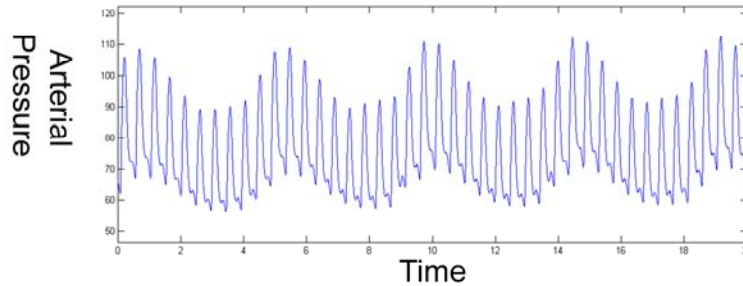

A phenomenon that is the reverse of the conventional pulsus paradoxus has been reported during positive pressure breathing

*F. Michard*

 Edwards Lifesciences

A phenomenon that is the reverse of the conventional pulsus paradoxus has been reported during positive pressure breathing.


The inspiratory increase in arterial blood pressure followed by a decrease on expiration has been called at different times:

- Reversed pulsus paradoxus
- Paradoxical pulsus paradoxus
- Respirator paradox
- Systolic pressure variation (SPV)
- Pulse pressure variation

At least 12 peer-reviewed English-language studies have demonstrated the usefulness of the respiratory variation in arterial pressure (or its surrogates) in answering a crucial clinical question; “Can we improve cardiac output and hence hemodynamics by giving fluid?”

**Michard F. Changes in Arterial Pressure During Mechanical Ventilation. *Anesthesiology* 2005; 103:419-28.**

## SVV and Mechanical Ventilation

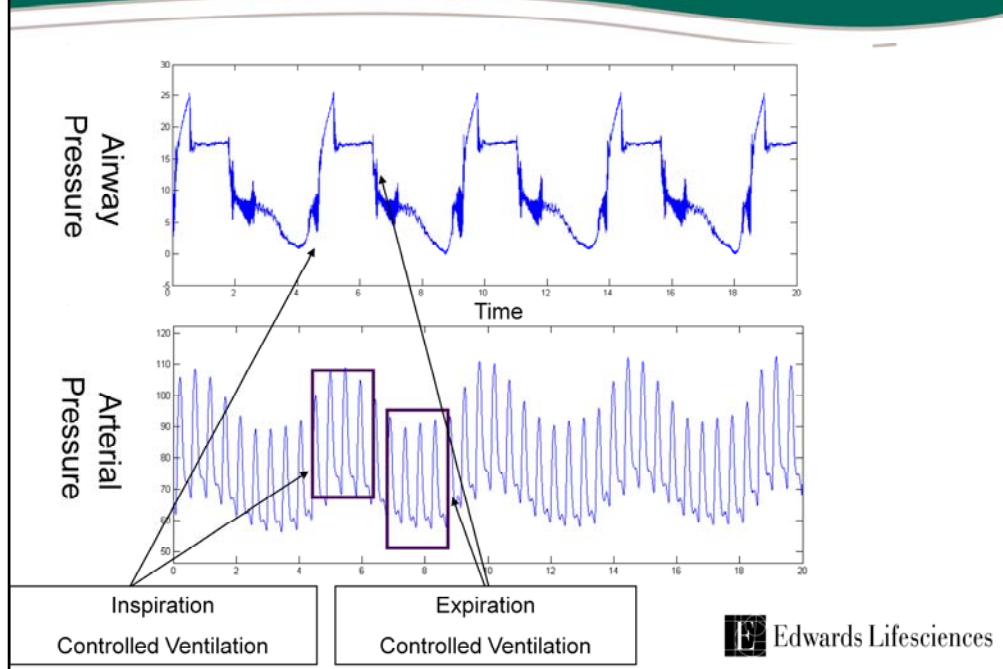


Mechanical ventilation induces cyclic changes in vena cava blood flow, pulmonary artery blood flow, and aortic blood flow. At the bedside, respiratory changes in aortic blood flow are reflected by “swings” in blood pressure whose magnitude is highly dependent on volume status. (Michard)

The left ventricular stroke volume increases during inspiration because left ventricular preload increases while left ventricular afterload decreases. In contrast, the right ventricular stroke volume decreases during inspiration because right ventricular preload decreases while right ventricular afterload increases. Because of the long (approximately 2 sec) pulmonary transit time of blood, the inspiratory decrease in right ventricular filling and output only a few heartbeats later, i.e., usually during the expiratory period. (Michard)

In healthy patients undergoing neurosurgery, a SVV value of 9.5% was found to predict a positive ( $\approx 5\%$ ) increase in stroke volume in response to only 100ml of plasma expander with a 79% sensitivity and 93% specificity. (Berkenstadt)

Currently, the majority of clinical literature supports the use of SVV in patients who are only mechanically ventilated.

## SVV as it is Traditionally Calculated

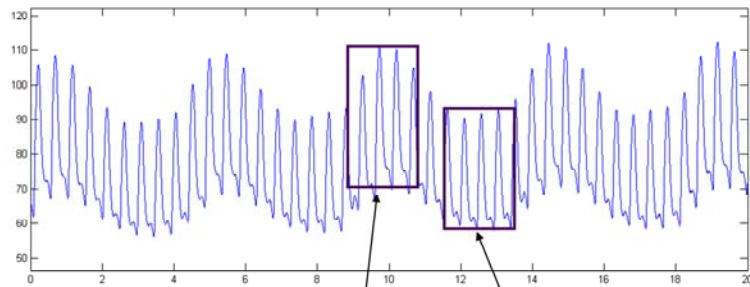

$$SVV = \frac{SV_{max} - SV_{min}}{SV_{mean}}$$

 Edwards Lifesciences

The above equation is the conventional method by which SVV is calculated. The difference between the maximum and minimum stroke volumes within a given time are divided by the mean of the maximum and minimum SVs. The SVV is displayed as a percent value.

## Sensitivity & Specificity of SVV

<u>Study</u>	<u>Patients</u>	<u>Vt ml/Kg</u>	<u>Parameters Tested (Artery)</u>	<u>R2</u>	<u>Def. of Responder</u>	<u>Sensitivity</u>	<u>Specificity</u>
Michard	Sepsis	8 to 12	$\Delta PP$ (R or F)	0.85	$\Delta CO \geq 15\%$	94	96
Berkenstadt et al.	Neurosurgery	10	$\Delta SVV$	0.53	$\Delta SV \geq 5\%$	79	93

\* Stroke Volume Variation correlates ( $\Delta SVV$ ) with Pulse Pressure Variation ( $\Delta PP$ )

With such high sensitivity and specificity the above studies demonstrate that SVV can answer the question:

“Can we use fluid to improve hemodynamics?”

Michard F, et. al. Relation between respiratory changes in arterial pulse pressure and fluid responsiveness in septic patients with acute circulatory failure. *Am J Respir Crit Care Med* 2000; 162:134-8

Berkenstadt H. Stroke Volume Variation as a Predictor of Fluid Responsiveness in Patients Undergoing Brain Surgery. *Anesth Analg* 2001; 92:984-9.


 Edwards Lifesciences

These two studies demonstrate that SVV has a high sensitivity and specificity in determining if a patient will respond (increasing stroke volume) when giving additional volume. This is referred to as “preload responsiveness”.

SVV values used to guide preload responsiveness have varied between 10 and 15%.

Using SVV allows us to answer the question “Can we use fluid to improve hemodynamics?”

## Application of SVV

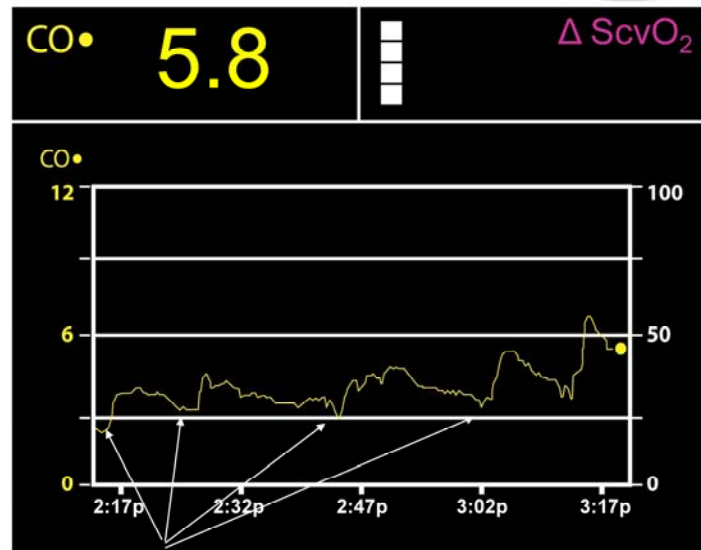


Volume resuscitation points

Edwards Lifesciences

The above case is an exploratory laparotomy with tumor removal. The FloTrac Sensor was used in addition to traditional vital signs.

The patient experienced a sudden loss of blood during the procedure and was volume resuscitated (packed red blood cells and Normal Saline). SVV was used as a guide for starting and stopping volume resuscitation at the points shown by the arrows. The following slides demonstrate the patient's response to resuscitation at the point highlighted by the second arrow above.


## Application of SVV

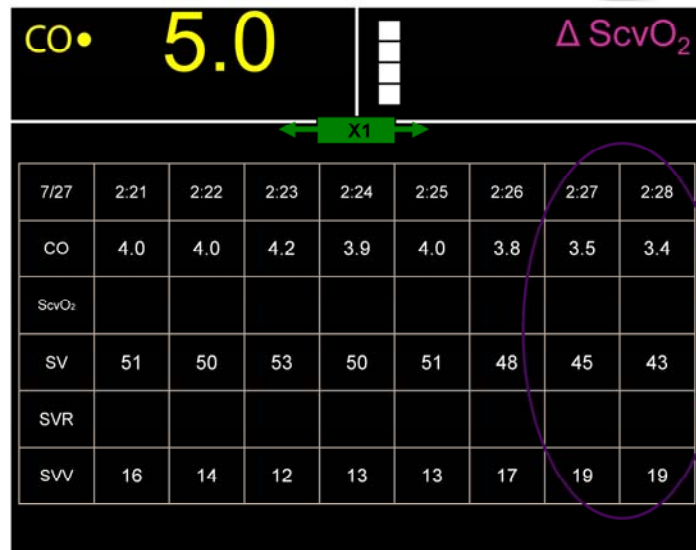

 Edwards Lifesciences

With a SVV of 19% and stroke volume of 45 ml/beat, the patient received one unit of packed red blood cells and 500ml Normal Saline.


## Application of SVV

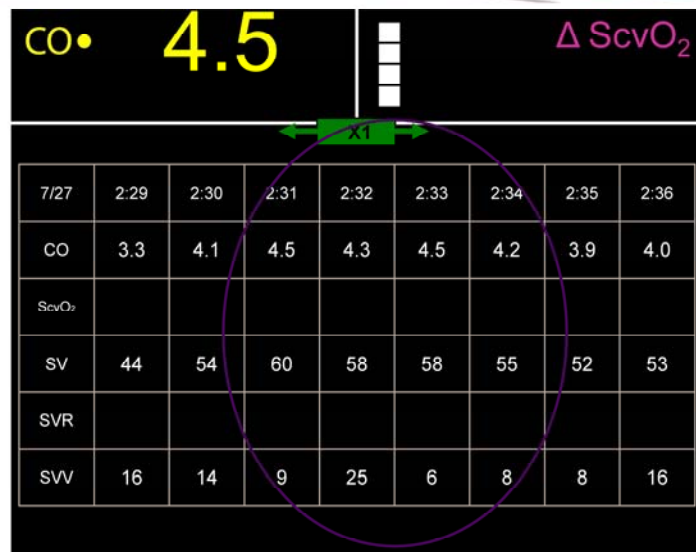

 Edwards Lifesciences

After resuscitation the SVV decreased to between 9 - 6% with a Stroke Volume increase to approximately 60 ml/beat. One aberrant reading of 25% (at 2:32 p) was due to an arrhythmia.

## Application of SVV

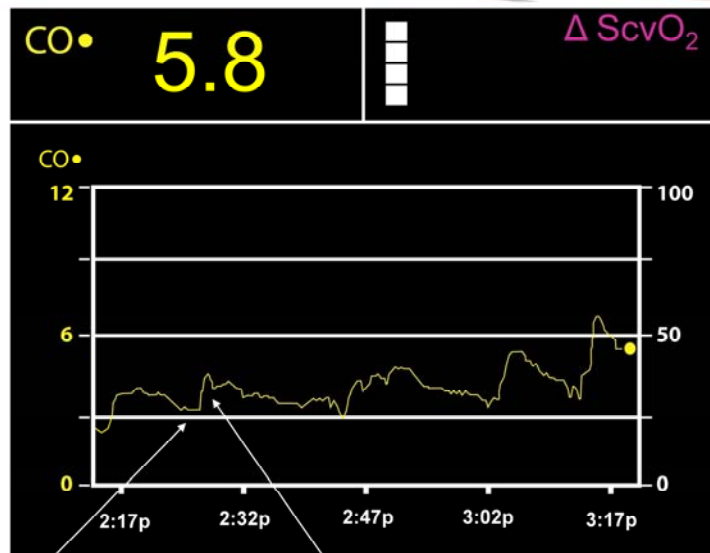

PRBC & NaCL given, SVV 18% and CO 3.4l/pm

SVV 9% & CO 4.5l/pm after infusion

Edwards Lifesciences

As displayed in the numerical trend screen, the graphical trend screen also shows an increase in cardiac output in response to volume resuscitation.

## What are the Limitations of SVV?

- **Mechanical Ventilation**
  - Currently, literature supports the use of SVV on patients who are 100% mechanically (control mode) ventilated with tidal volumes of more than 8cc/kg and fixed respiratory rates.
- **Spontaneous Ventilation**
  - Currently, literature does not support the use of SVV with patients who are spontaneously breathing.
- **Arrhythmias**
  - Arrhythmias can dramatically affect SVV. Thus, SVVs utility as a guide for volume resuscitation is greatest in absence of arrhythmias.



Although a powerful tool managing your patients volume resuscitation, SVV has limitations.

### Mechanical Ventilation:

Current literature supports the use of SVV on patients who are 100% mechanically (control mode) ventilated with tidal volumes of more than 8cc/kg and fixed respiratory rates.

### Spontaneous Ventilation:

Currently, literature does not support the use of SVV with patients who are spontaneously breathing. Spontaneous breaths uses negative pressure ventilation with small, varying tidal volumes.

### Arrhythmias:

Arrhythmias can dramatically affect SVV. Thus, SVVs utility as a guide for volume resuscitation is greatest in absence of arrhythmias.

### SVR:

The effects of vasodilatation therapy on SVV should be considered before treatment with additional volume

## Positive End Expiratory Pressure (PEEP) and SVV

- **PEEP may decrease cardiac output and offset the expected benefits of oxygen delivery**
- **Arterial waveform analysis is useful to predict and prevent the deleterious hemodynamic effects of PEEP in mechanical ventilation**



The deleterious hemodynamic effects of PEEP are caused by an increase in pleural pressure (reducing right ventricular filling) and an increase in Transpulmonary pressure (increasing right ventricular afterload).

When cardiac output decreases with PEEP the arterial pressure variation increases. If PEEP does not affect cardiac output the arterial pressure variation is similarly unaffected by PEEP.

The arterial pressure waveform analysis is useful to predict and prevent the deleterious effects of PEEP in mechanical ventilation.

**Michard F. Changes in Arterial Pressure During Mechanical Ventilation. *Anesthesiology* 2005; 103:419-28.**

## Summary

- **SVV is easily measured and displayed at the bedside with the Vigileo monitor**
- **Studies reviewed by Michard demonstrate that dynamic indicators such as SVV and PPV are effective in determining the patient's fluid responsiveness, while static indicators of preload (CVP, PAD, PAOP) are frequently unable to correctly answer an important question...**

**“Can we use fluid to improve hemodynamics?”**



Edwards Vigileo monitor when used with the FloTrac Sensor, easily measures and displays SVV at the bedside.

SVV is a very reliable indicator of the patients preload responsiveness with a high sensitivity and specificity

When used within its limitations, it is a powerful tool that can correctly answer that important question.....

“Can we use fluid to improve hemodynamics”


## Bibliography

- Berkenstadt H. et. al. Stroke Volume Variation as a Predictor of Fluid Responsiveness in Patients Undergoing Brain Surgery. *Anesth Analg* 2001; 92:984-9.
- Michard F. Changes in Arterial Pressure During Mechanical Ventilation. *Anesthesiology* 2005; 103:419-28.
- Parry-Jones AJD et. al. Arterial Pressure and Stroke Volume Variability as Measurements for Cardiovascular Optimization. *International Journal of Intensive Care*; Summer, 2003; 67-72.
- Teboul JL. Arterial Pulse Pressure Variation During Positive Pressure Ventilation and Passive Leg Raising. In: M.R. Pinsky, D. Payen, eds. *Functional hemodynamic monitoring: Update in intensive care and emergency medicine* 42. Berlin: Springer, 2005; 331.
- Swami A. et. al. Pulsus Paradoxus in Cardiac Tamponade: A Pathophysiologic Continuum. *Clin. Cardiol.* 2003; 26:215-217

Rx only. See instructions for use for full prescribing information.  
Edwards Lifesciences devices placed on the European market meeting the essential requirements referred to in Article 3 of the Medical Device Directive 93/42/EEC bear the CE marking of conformity.  
Edwards, Life is now and Vigileo are trademarks of Edwards Lifesciences Corporation. Edwards Lifesciences, the stylized E logo, FloTrac and PreSep are trademarks of Edwards Lifesciences Corporation and are registered in the United States Patent and Trademark Office.  
© 2008 Edwards Lifesciences LLC  
All rights reserved. AR03089



Edwards Lifesciences LLC · One Edwards Way · Irvine, CA 92614 USA · 949.250.2500 · 800.424.3278 · [www.edwards.com](http://www.edwards.com)



Edwards

Helping patients is our life's work, and *life is now*

